

Abscess Remedy Paste 12g



Root-canal radiopaque and bactericidal treatment paste

PROPERTIES

Abscess Remedy Paste is a radiopaque paste which combines bactericidal and antiseptic effects. It is used as a temporary treatment of the root-canal.

INDICATIONS

- Treatment of infected 4th degree caries
- Temporary filling of the root-canal.



COMPOSITION

Powder : Paraformaldehyde 1.1%, radiopaque excipient ad 100%

Liquid : Formaldehyde (40% vol.) 43.8%, Beechwood Creosote 9.2%, Thymol 4.9%, excipient ad 100%

Directions to Use:

- After proper cleansing, shaping and irrigation of the root-canal, 1 application of the paste kept on place during at least 1 week, should relief pain and ensure complete disinfection. If required repeat the treatment 1 to 2 times at several days interval.
- Work the paste deeply into the root-canal with the aid of a smooth broach or a rotary paste filler. Avoid air pockets or extrusion beyond the apex. A permanent filling can be applied after eliminating of the last applied paste as well as shaping and re-treatment of the canal.
- After pulpectomy, obturate the root-canal with "PD" Abscess Remedy Paste. During the next seat, the paste is to be removed and the mechanical and chemical cleansing of the root-canal is to be achieved. Then proceed as per the traditional technique for the permanent obturation of the root-canal.