

Endofill (15g+15ml)

Root-Canal filling material



INDICATION

- Radiopaque paste with Dexamethasone for the permanent obturation of the root-canal

PROPERTIES

Endofill is a radiopaque preparation for permanent root-canal filling. Its composition is well tolerated by tissues and it provides anti-inflammatory, antiseptic and germicidal actions. Before hardening, the paste penetrates the narrowest fissures and maintains its therapeutic effects throughout the treatment until completely set. The final obturation neither retracts nor resorbs.

COMPOSITION

Powder :

Dexamethasone Acetate 0.01%, Hydrocortisone Acetate 1.0%, Polyoxymethylene 2.2%, Thymol Iodide 22.5%, excipient ad 100%

Liquid : Eugenol



Directions for use:

- Clean the canal thoroughly and dry with hot air
- Mix the powder and liquid until to obtain a paste of the required consistency (see list)
- Work the paste into the root-canal with the aid of a probe or a rotary paste filler
- Fill the canal completely, avoid air pockets, and do not protrude beyond the apex.