

Pulpofluorane- Pulp Vitality Tester, 235ml



Behind Every Dentist

Indications:

- Pulp vitality tester
- Cooling of mixing slabs

Directions for Use:

- Hold a cotton wool roll/pellet with a pair of tweezers. Spray some pulpofluorane on to one end of the cotton roll or on the whole surface of the cotton wool pellet.
- Apply the refrigerated surface of the cotton wool or pellet on to the surface of the tooth to be tested, applying pressure for 5 seconds, and then continue by using short touches until a slight sensation of pain occurs.
- The cooling of the mixing slabs is reached in a few seconds by a simple spray.

Composition: (1) Tetrafluoroethane 96.6% (2) Dimethylether 3.4 %

